

## Clinician's Reference Card





### Programming Set-Up

1. Log into the StimRouter Clinician's Software.
2. Connect the Patient Programmer to the Clinician Programmer via the Programmer Connector Cable.
3. Create a new patient record or open an existing patient record.

### StimRouter Electrode/External Pulse Transmitter (EPT) Set-Up

1. After the EPT is charged, snap the EPT onto the StimRouter Electrode.
2. Remove the electrode liner and store it in the StimRouter Electrode Carrying Case.
3. Grasp the edges of the EPT attached to the StimRouter Electrode so the gel pads face down.
4. Adhere the end of the StimRouter Electrode, that has the EPT charging port, directly over the receiver end of the lead.

**Note:** The center of the gel pad should be above the receiver end of the lead.

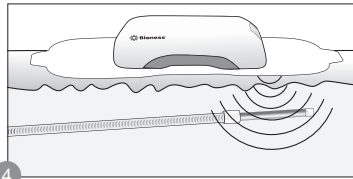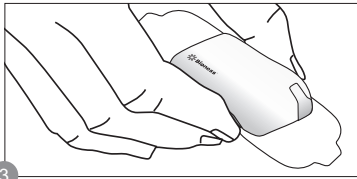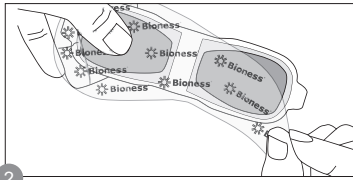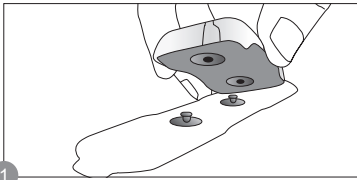

### Software Navigation

Settings Tabs ———

Program Bar ———

Intensity Level Bar ———

Menu Bar ———

Save Icon

Print Icon


Add Program

Delete Program

# Programming


## Programming Stimulation Parameters:

1. From the Patients Menu open a patient record to view the Stim Settings Tab.
2. Press the “Test” button to test the stimulation parameters.
3. If needed adjust the intensity level and fine tune the stimulation parameters using the drop-down lists.
4. Press the “Stop & Save” button to stop testing and save the current stimulation parameters.

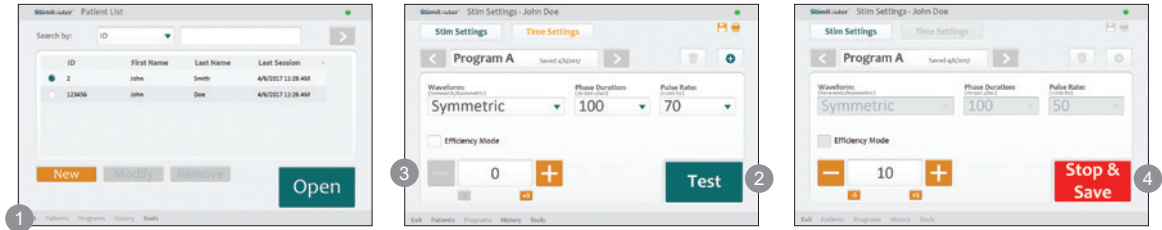


## Programming Time Settings:

1. Press the Time Settings Tab.
2. Adjust the time settings using the drop-down lists. To adjust the “Time On” and “Time Off” settings uncheck the “Constant Stim” box.
3. Press the “Test” button to test the stimulation time settings. If needed adjust the intensity level.
4. Press the “Stop & Save” button to stop testing and save the current stimulation time settings.

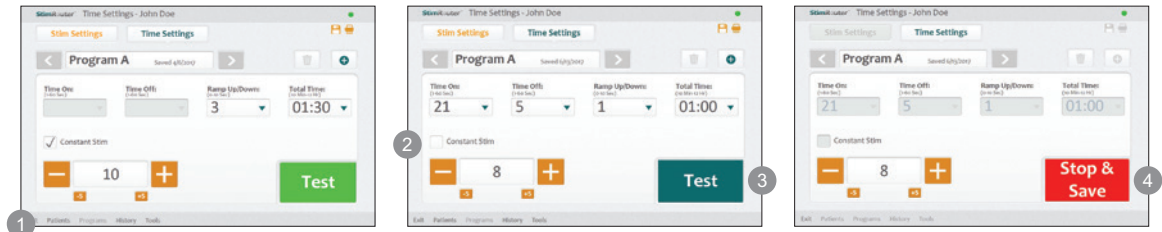

## Adding a Program

1. From the Stim Settings Tab or the Time Settings Tab press the add program icon on the program bar.

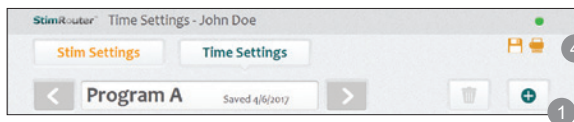

2. A new program will open. The StimRouter software can support up to eight stimulation programs, labeled (A-H).
3. Program the settings, test the settings and press the “Stop & Save” button to save the new settings.
4. To print the new program press the print icon on the program bar.